

# CRYSTAL PEELING EXCELLENCE

## CLINICAL STUDY by Dr De Goursac (France)

Dermabrasion is a technique, which consists in removing the superficial layers of the skin with a stream of micro-crystals to stimulate the skin renewal.

This treatment can be superficial to deep, it depends on:

- >> Flow and density of the Stream of crystals
- >> Time of treatment at the same place

### Indications

- Scars
- Pigmentation disorders
- Wrinkles
- Stretch marks
- Flaccidity
- Sun damages


Following treatments have been realized with the Oxynergy Paris Crystal peeling excellence device by Dr De GOURSAC (France)



- Remove Small scars after 5 sessions
- Important improvement of deep scars

Number of sessions depends on the scars

**After 4 treatments** / 10 days between treatments

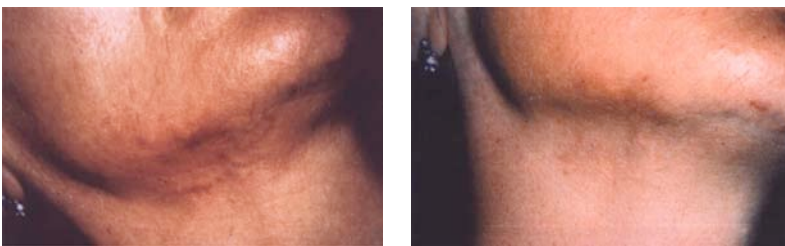

### Acne keratosis

2 to 3 treatments can remove most of the acne keratosis. Depth of abrasion must be adjusted to the hyperpigmentation localisation.




## IMPORTANT POINTS

- Protect the eyes of the patient during treatment
- Maintain the skin tight with the fingers while performing the abrasion
- Never press the applicator against the skin
- Stop abrasion if bleeding
- Always work on a cleansed skin

We recommend to apply collagen mask Oxynergy Paris and cryo to sooth the skin.

How to perform the treatment:

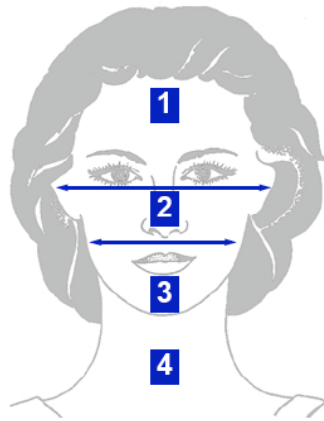

The face is divided in different areas

1. Upper
2. Mid
3. Lower
4. Neck

