



OXYNERGY ENHANCER ®
NO NEEDLE MESOTHERAPY

Description and Operating Instructions

E-mail: info@oxynergy.com
Website: www.oxynergy.com

Oxynergy Enhancer ®

1. Legend
2. Description
 - 2.1 Purpose
 - 2.2 Function
3. Safety Information
 - 3.1 Precautions
 - 3.2 Trouble shooting
 - 3.3 Indications
4. Installation
5. Programs
 - 5.1 Low
 - 5.2 Medium
 - 5.3 High
6. The applicators
7. Technical data
 - 7.1 General data
 - 7.2 Maintenance
8. Guarantee

Oxynergy Enhancer ®

GENERAL WARNING:

- The Oxynergy Enhancer must always be plugged on a switch with earthing
- The cables of the headpieces must be connected or disconnected always when the device is not under tension
- The cables must never be disposed over the screen of the device or the buttons in operation or some perturbations can be observed on the program

1. Legend

Buttons and indicators

LCD screen indicating the treatment, the power and the length of the treatment.

On/Off: Start or stop the treatment

Low: Facial treatment for sensitive skin

Medium: Facial treatment for all skin types

High: Body treatment

2. Description

2.1 Purpose

The Oxynergy enhancer is a new technology of transdermal administration of all kind of substances without needle and any kind of electric shock. It's indicated in all kind of applications like the treatment of flaccidity, cellulite, sport medicine etc. As indicated before this unique system of electroporation permit to introduce directly in the heart of the cells all kind of substances (ionic, non ionic, lipophilic or hydrophilic). It appear to be two different effective pathways in clinical studies performed on similar system: cutaneous and transfollicular delivery. Through exclusive non ionizing and athermic electromagnetic waves, the Oxynergy enhancer creates a biologic stress of the skin cells and a partial destabilization of the cell membrane potential that increase of 400 times its permeability to all kind of substances. This phenomenon is reversible and have a short lasting as demonstrated in various publications. The absorption speed is 1 g per minute for emulsions and higher for lipophilic products). The Oxynergy Enhancer does not present any kind of risk of burn, electric discharge, inflammation and is 100 % painless. It's also a small device that can be very easily moved or transported by therapists.

The Oxynergy Enhancer can be used as a single treatment with all the skin care line but also to enhance the resulted reached in Oxygen therapy and complement the Oxynergy lift technique.

2.2 Function

We apply non ionizing and athermic electromagnetic waves, to create a biologic stress of the skin cells and a partial destabilization of the cell membrane potential that increase of 400 times its permeability to all kind of substances. This phenomenon is reversible and have short lasting.

3. Safety Information

3.1 Precautions

- Patient with pace maker.
- Pregnancy
- Epileptic (if the treatment is made on the head or the neck).
- Patient suffering skin degenerative process.
- Any kind of infected source.

- After a peeling it is mandatory that the skin has recovered completely.
- Abdominal area during menstruation
- Phlebitis in varicose area.

3.2 Trouble shooting

CALL SERVICE

3.3 Indications

- AESTHETIC:
 - Obesity & cellulite.
 - Flaccidity.
 - Skin rejuvenation.
 - Scars.
 - Stretch marks.
- MEDICAL:
 - Rheumatologic and neuromuscular alteration
 - Osteoarticular irregular and chronic pathologies.
 - Consequences of muscular traumatism and breakings.
 - Neuralgias.

4. Installation

Plug the devise with the cable furnished with the equipment to the sector.

Checks that the cable is connect at the back of the equipment.

Connect first the desired applicator to the device (facial or corporal)

Turn on the device

When turning on the devise the LCD screen will show

“Welcome Oxynergy”

Then the treatment menu

Choose one of the programs with the buttons LOW – MEDIUM – HIGH and push on/off to start the treatment.

5. Programs

5.1 Low

On the LOW mode the device is furnishing 30 % of its potency. It's adapted to work on the face for sensitive skin.

5.2 Medium

On the MEDIUM mode the device is furnishing 60 % of its potency. It's adapted to work on the face for all skin types.

5.3 High

On the HIGH mode the device is furnishing 100 % of its potency. It's adapted to work on the body for all skin types or on the face in the eventually you are working with creams or macromolecules.

6. The applicator

2 different applicators are furnished with the device. One for the body and one for the face. Always turn off completely the device to change the applicator. Clean the applicators with alcohol between each patient.

7. Technical data

7.1 General data

Power supply: 110–220 V /50–60 Hz
Power absorption: 120 W
Command management: by microprocessor
Display: LCD 2x20
Alpha numeric character negative bleu
Digital management
Maximum sound level: 3 dB
Case size: l: 200 mm h: 100 mm d: 200 mm
Weight: 2 Kg
Use temperature: + 10 of + 40 °C
Humidity relative: 30 of 75 %
Security: Class 1 type BT (norm NF EN 60335–1)

7.2 Maintenance

Return to service every 5 years.
Change the applicators when used.

8. Warrantee

This devise must be handle by a qualified person. Oxynergy is not responsible of damages to person due to an inappropriate use of the devise. The Devise must be plugged only on adapted location (16 A)

The devise is guarantee for 5 years from the invoice day for any default of manufacturing. The transport costs are due by the client. This warranty does not cover damages from accidents, inappropriate use or if the devise have been opened or modified by third person.